

Aerogen® Solo

Continuous Nebulisation

Quick Set-up Guide



At the Wye



Dry Side of the Humidifier

- 1 Remove the syringe cap from the medication filled syringe and attach the syringe end of the tubing onto the syringe.



- 2 Prime the tubing until the medication reaches the end of the tubing. (Maximum priming volume is 3.65mL)



- 3 Screw the nebuliser end of the tubing onto the top of the nebuliser.



- 4 Press and hold the on/off button for 3 seconds from the off position to select the continuous mode.



Note: Humidifier parts not supplied by Aerogen. Consult manufacturer for usage guidelines.



Please note:

- Aerogen's recommended input rate of medication into the Aerogen Solo during continuous nebulisation is up to a maximum of 12 mL per hour. The upper limit of 12 mL per hour is based on Aerogen's specification for the minimum flow rate.
- Replace both the tube set and syringe when changing the type of medication.
- If the syringe needs to be replaced for any reason including when empty, always disconnect the Solo end first. Failure to do so may result in the flow of medication, already in the syringe tubing, into the Aerogen Solo.
- To avoid spillage of medication when changing the syringe tubing, keep both ends of the tubing at the same height.