HUN /

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ENG /
<p>Aerogen® Ultra is a accessory specific to the Aerogen Solo nebuliser. It facilitates intermittent and continuous nebulisation, with optional supply of supplemental oxygen to paediatric and adult patients via mouthpiece. The device can alternatively be used with a face mask, which is not provided.</p> <p>It is a single patient use device which is qualified for 20 intermittent use treatments (at a rate of four 3 ml doses per 5 days) or 3 hours of continuous use.</p> <p>For continuous use, the Aerogen Ultra can be used in conjunction with the Aerogen Solo Continuous Nebulization Tube Set. See Aerogen Solo System Instruction Manual.</p> <p>Optimal aerosol delivery is achieved with valved mouthpiece or valved aerosol face mask with low/no oxygen flow.</p> <p>Aerogen Ultra Kits</p> <p>AG-AS7500-EU: Aerogen Ultra Starter Kit (10 pack): Aerogen Solo, Aerogen Ultra, oxygen tubing</p> <p>AG-AS7000-EU: Aerogen Ultra (10 pack): Aerogen Ultra, oxygen tubing</p> <p>Note: Device names previously referencing “Aeroneb” are now referencing “Aerogen”. The Aerogen Ultra was also previously known as Aeroneb Solo Adapter.</p> <p>Instructions for Use – Refer to Figures 1 - 8</p> <p>Inspect for device integrity and correct valve placement prior to use.</p> <ol style="list-style-type: none">Insert Aerogen Solo nebuliser firmly into Aerogen Ultra in orientation shown. If supplemental oxygen is required, firmly attach oxygen tubing to Aerogen Ultra. Note: Oxygen flow rate should be set between 1-6 LPM. If a face mask is required, remove mouthpiece and attach face mask to Aerogen Ultra. Note: When using an open face mask, a minimum oxygen flow of 1 LPM is required. Add prescribed medication to nebuliser as instructed in Aerogen Solo System Instruction Manual. Connect cable to Aerogen Solo. Note: Refer to Aerogen Solo System Instruction Manual for intermittent and continuous nebulisation set-up and operation instructions. Power on and observe Aerogen Ultra to confirm aerosol is visible. Introduce Aerogen Ultra to patient and observe aerosol flow to ensure correct operation. Remove excess runoff from the Aerogen Ultra periodically (hourly) with continuous nebulisation). To ensure optimum performance of the Aerogen Ultra, remove any residue by rinsing through with sterile water, shake off excess and allow to air dry.

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AG-AS7500-EU: Zestaw startowy Aerogen Solo (10 szt.); Aerogen Solo, Aerogen Ultra, dren do podawania tlenu
AG-AS7000-EU: Aerogen Ultra (10 szt.); Aerogen Ultra, dren do podawania tlenu